BH

# Competence universal®

## Uputstvo za upotrebu

Competence universal je svjetlosnoštrajnjavjući , neprozirni univerzalni kompozit za punjenje koji se koristi za obnovu svih klasa zubnih kanala. Pomoću njegove specifične sposobnosti modeliranja on je posebno prikladan za višeslojnu i višebojnu tehniku. To ne samo da garantuje prirodno i visoko estetsko punjenje već i minimalno smanjenje polimerizacije.

### Indikacije

Obnova svih zubnih kanala

### Preporučena procedura

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# Competence universal®

## Instructions for use

Competence universal is a light curing, x-ray opaque universal filling composite for restorations of all cavity classes. Its special modelling proper- ties make it particularly suited for the multiple shade layer technique. This not only guarantees a natural and highly aesthetic filling, but also minimisation of polymerisation shrinkage.

### Indication

Restoration of all cavity classes

### Recommended usage

1. Prior to preparation, clean the tooth, remove all cleaning residues with water spray. Determine the Vita® tooth shade. Use a rubber- dam where possible.
2. Prepare the cavity according to the general rules of adhesive technology. Round off enamel edges and protect pulp areas.
3. Etch as required, according to your preferred technique, e.g. Total-etch. Follow the instructions for use of the materials used.
4. Apply bond primer. (e.g. C-Bond)
5. Apply a lining of thin flowing composite (Competence Flow) or compomer as required. We recommend using a matrix.
6. Fill the cavity in the required shade in layers of maximum 2 mm thickness. Light-cure each layer for 20 seconds.
7. Surplus removal, contouring and fine shaping are carried out e.g. with a diamond finisher and flexible discs of decreasing grain size. High-gloss polishing can be carried out with a polishing paste (e.g. Vision Paste) using polishing cups /discs.

### Caution

- Light devices should emit at 450 nm and be regularly inspected. The light should be placed as close as possible to the filling material.
- Do not remove the oxygen-inhibited layer, as this is required for the bond with the next layer.
- Follow the instructions for use of the other materials used with Competence universal.

### Interactions

Avoid materials containing eugenol, moisture and airborne oil – they can prevent the polymeri- sation of the contact area. Use of cationic mouth rinses, plaque revelators or chlorhexidine can cause discolouration.

### Contraindications

The application of Competence universal is contraindicated if drainage or the recommended application technique cannot be achieved. Do not apply directly on open pulp or if there are any known allergies against one of the components.

### Side effects

No side effects are known to date. Hypersensitivity againts components of the material cannot be excluded in every case. Avoid skin contact with the paste. In the event of inadvertent contact, wash immediately with soap and water. In the event of eye contact, immediately rinse with plenty of water and seek medical treatment. If you experience any undesired effects, even in case of doubt, we ask for notification in every

case, describing the concomitants and symptoms as detailed as possible.

### Processing instructions

Observe the processing instructions in the instructions for use included with the product and the Precautionary Measures under all circumstances. Should the patient still show an undesired reaction or if there are undesired effects, we ask for notification in every case, describing the concomitants as detailed as possible.

### Composition

Dental glass in a BIS-GMA based matrix of dental resins (oligomere Dimethacrylates), pigments, additives, catalyst. Filling material factor: 76 wt.% = 57 vol.%(0,02 – 1,5 µm).

### Classification

Competence universal complies with the requirements of DIN EN ISO 4049.

### Storage

Do not store above 25°C. Discharge syringes immediately after use to avoid the material curing from ambient light. Do not use after the expiry date.

### NOTE

Applicator-tools (tips, brushes) are only determinate for single-use at only one patient.

### Packaging

Competence universal, set 4 syringes

Competence universal, set 4 syringes

Competence universal, set 4 syringes

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